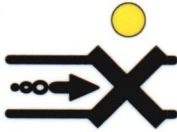
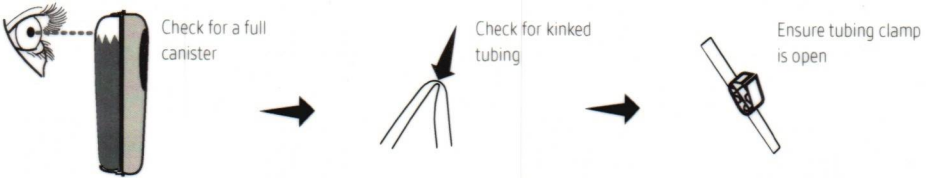

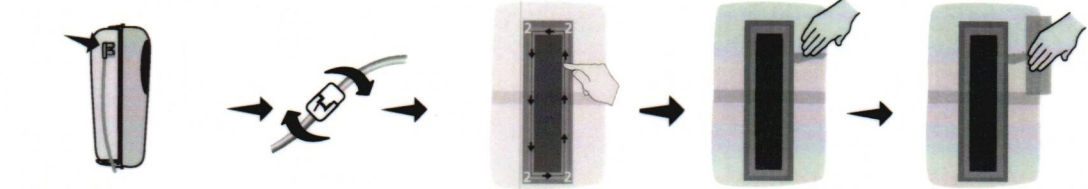

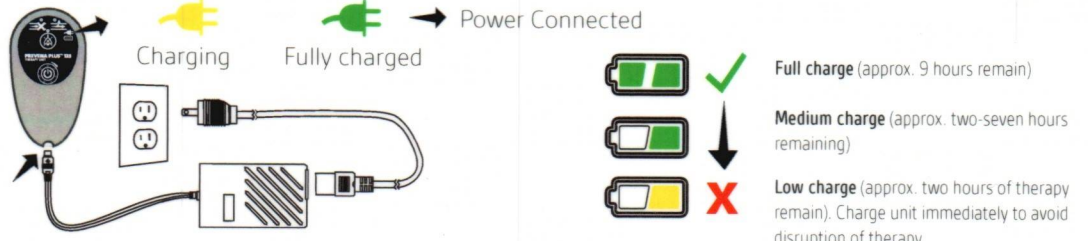

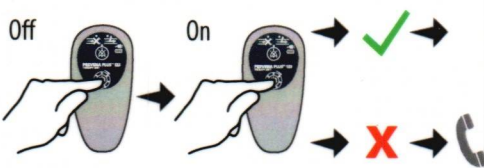




PREVENA PLUS™

INCISION MANAGEMENT SYSTEM



QUICK GUIDE

Alert Type	How to fix	
<p>Blockage Alert</p>  <p>Audible blockage alert will sound two beeps repeating every 15 seconds</p>	 <p>Check for a full canister</p> <p>Check for kinked tubing</p> <p>Ensure tubing clamp is open</p>	
<p>Leak Alert</p>  <p>Leak Alert will sound two beeps repeating every 15 seconds</p>	 <p>Ensure canister is security locked onto the therapy unit. When canister is installed, a distinct click will be heard.</p> <p>Check tubing connectors to ensure they are fully engaged and locked.</p> <p>Press down through drape onto the adhesive seal all the way around to ensure seal</p> <p>Use fingers to smooth out any creases or wrinkles</p> <p>Use PREVENA™ Patch Strips or excess KCI™ drape along the outer edge of drape to seal leaks</p>	
<p>Low Battery Alert</p>  <p>Alert will sound two beeps repeating every four minutes</p>	 <p>Charging Fully charged Power Connected</p> <p>Full charge (approx. 9 hours remain)</p> <p>Medium charge (approx. two-seven hours remaining)</p> <p>Low charge (approx. two hours of therapy remain). Charge unit immediately to avoid disruption of therapy.</p>	
<p>System Fault Alert</p>  <p>All LEDs flashing and two beeps every 15 seconds</p>	<p>Turn therapy unit off then on</p>  <p>Off On</p> <p>Continue therapy</p> <p>Call KCI at 1-800-275-4524</p>	
<p>Alert Mute Button</p> 	<p>Press and hold center of the alert mute button for three seconds during an alert condition to silence the audible alert for two minutes. When pressed, the alert mute button will illuminate to indicate mute has been selected. The alert will re-occur after two minutes unless the alert condition has been corrected</p>	
<p>Therapy Ended</p> 	<ul style="list-style-type: none"> A solid yellow LED at the top of the Therapy Life Indicator will turn on. The therapy unit will sound eight beeps, followed by a continuous beep for five seconds, then the therapy unit will turn off. Notify the treating physician or caregiver: If the therapy unit has completed the seven day therapy and has timed out, and an attempt is made to turn the therapy on, the therapy unit will sound an alert for three seconds then shut off 	

NOTE: Specific indications, contraindications, warnings, precautions and safety information exist for the PREVENA™ Therapy. Please consult the applicable PREVENA™ System Clinician Guide instructions for use prior to application. Rx only.

Copyright 2017 KCI Licensing, Inc. All rights reserved. All trademarks designated herein are proprietary to KCI Licensing, Inc., its affiliates and/or licensors. PRA001264-RO-US, EN • LIT#29-B-272 (08/17)